

**Blake 13/Ellison 13/Blake 14
Panda iRes Warmer**

Updated

9/10/2019

Software	Rev
Main Application	2.0.5
Service Application	2.1.5
Heat Processor	1.5.4
Audio Processor	2.0.0
Alarm Processor	1.5

Feature	Available Settings	MFR Defaults	MGH Defaults
Patient temp. alarm	0.5°C / 1.0°C	1.0°C	1.0°C
Check patient alarm disabled	5% - 50%	25%	25%
Man. temp. alarm	Off / 0.5°C / 1.0°C	Off	Off
Alarm volume	1 / 2 / 3 / 4	4	3
Hands free silence	Enable/ Disable/ Disengaged	Enable	Enable
Bed elevate (for Bedded Warmers with Elevating Base)	Enable/ Disable/ Disengaged	Enable	Enable
Bed elevate (for Bedded Warmers with Fixed-Adjustable Base, Freestanding armers, and Wall Mount Warmers)	Enable/ Disable/ Disengaged	Enable	Enable
Temp. units	C / C Only / F	C	C
Scale units	g/ lb/ g only	g	g
Scale resolution	10/5 g	10 g	10 g
Language	Many	English	English
Low pulse rate limit	25 - 110	100	100
High pulse rate limit	111-240	200	200
Low saturation limit	20 -99	85	85
High saturation limit	86-100	100	100
SpO2 alarm suspension on startup	0 / 15 / 30	15	15
SpO2 averaging	2-4 / 8 / 16	8	8
SpO2 sensitivity	Normal / Maximum	Normal	Normal
SpO2 Satseconds	Off / 10/25/50/100	off	off
SpO2 view	Large/ Normal	Large	Large
Pulse tone volume	0 / 1 / 2 / 3 / 4	1	1
Time Format	12 / 24	12	24
Date Format	YYYY/MM/DD	MM/DD/YYYY	MM/DD/YYYY
	MM/DD/YYYY		
	DD/MM/YYYY		