



# Zoll AED Plus Defibrillator Weekly Checklist



Year \_\_\_\_\_

Step	Check Daily:		Week No.	Signature	Comments
1	(If in cabinet) Verify white cabinet is closed and has no damage (no cracks in glass, etc...)		1		
			2		
			3		
2	Verify that the date on the green tab has not passed. If the date on the green tab is less than 30 days away, contact your operations supervisor to order new CPR-D Padz		4		
			5		
			6		
			7		
			8		
			9		
3	Verify the Status Indicator is a Green ✓ If Green ✓, test is finished. If <b>not</b> a Green ✓, go directly to step 5		10		
			11		
			12		
			13		
			14		
4	Sign your name on the appropriate space to the right. Add comments as necessary.		15		
			16		
			17		
			18		
			19		
			20		
5	Contact Biomedical Engineering at 617-732-8889 immediately for service.		21		
			22		
			23		
			24		
			25		
			26		
			27		
			28		
			29		
			30		
			31		
			32		
			33		
			34		
			35		
			36		
			37		
			38		
			39		
			40		
			41		
			42		
			43		
			44		
			45		
			46		
			47		
			48		
			49		
			50		
			51		
			52		